

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings



Description:

ESPUMA® Gentle is absorbent wound dressing made of polyurethane foam .The outermost layer is bacteria-proof, virus-proof and water-proof polyurethane film. The wound contact layer is perforated soft silicone gel that allows painless and atraumatic removal and permits the passage of exudate to the absorbent foam layer.

ESPUMA® Gentle Lite consists of thin absorbent polyurethane foam layer.

ESPUMA® Gentle Comfort consists of absorbent foam layer having an adhesive outermost polyurethane border, so the dressing can be applied without fixation tape or secondary dressing.

ESPUMA® Gentle Lite Comfort has a thin foam layer with an adhesive outermost PU border.

ESPUMA® Gentle C Comfort is an absorbent foam with additional non-woven layer containing activated carbon to adsorb offensive odour and with an adhesive outermost PU border.

ESPUMA® Gentle Lite C Comfort differs from the previous variant in having a thinner polyurethane foam layer.

ESPUMA® Gentle Xtra Comfort is a multi-layered dressing consisting of absorbent foam with additional highly absorbent layer and having an adhesive outermost border.

ESPUMA® Gentle Lite Xtra Comfort differs from the previous variant in having a thinner polyurethane foam layer.

ESPUMA® Gentle C Xtra Comfort is a multi-layered dressing consisting of absorbent foam, with additional highly absorbent layer plus a non-woven layer containing activated carbon to adsorb offensive odour and having an adhesive outermost border.

ESPUMA® Gentle Lite C Xtra Comfort differs from the previous variant in having a thinner polyurethane foam layer.

ESPUMA® Gentle Transfer is composed of thin polyurethane foam layer, but without polyurethane top sheet, so that the dressing can transfer exudates away from the wound into a secondary absorbent dressing.

Features:

- Comfortable: as the dressing is easy to remove and minimize trauma during dressing changes.
- Cushioning effect: as it acts as a barrier against irritation and constant pressure on the skin.
- Breathable: the semi-permeable backing allows moisture to escape.
- Prevents skin maceration: as the perforated silicone film allows free passage of wound exudate to the foam.
- Additional carbon layer in some variants: adsorbs offensive odour in case of malodorous wounds.
- Additional superabsorbent layer in some variants: increases the absorption capacity of the dressing for highly exuding wounds.

Intended Purpose:




ESPUMA® Gentle Family is an effective tool for moist wound healing, having high capacity of autolytic debridement and it is particularly useful in preventing dressing-related trauma especially in people with fragile skin. It's indicated for a wide range of exuding wounds as: pressure ulcers, leg ulcers, diabetic foot ulcers, skin graft, first and second degree burns. Can be used in combination with compression therapy.

The Lite variants are indicated for low exuding and traumatic wounds as: abrasions, blisters and skin tears.

The variants containing carbon are for malodorous acute and chronic wounds as carbon adsorb offensive odor.

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

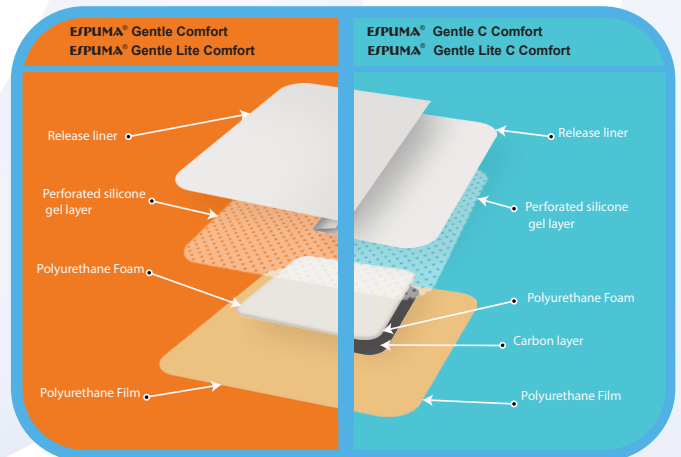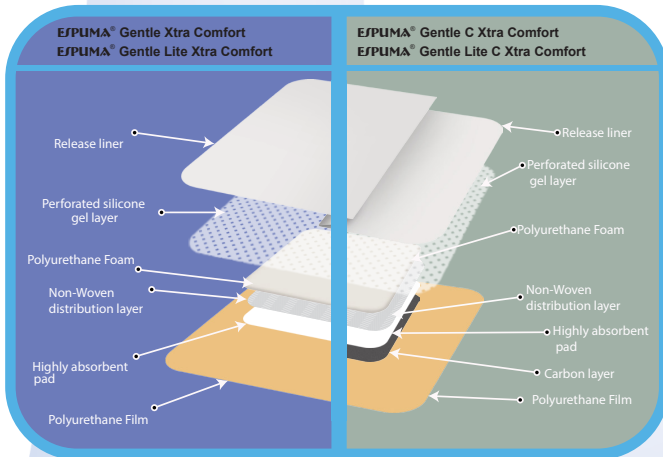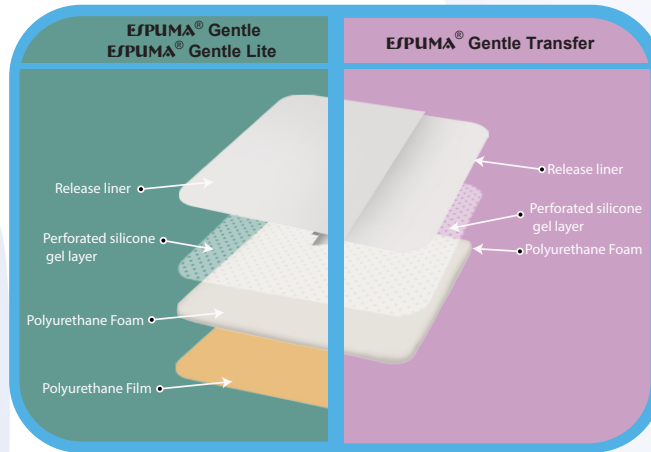

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPG7575	75 x 75 mm	10	48	
ESPG75100	75 x 100 mm	10	48	
ESPG100100	100 x 100 mm	10	48	
ESPG150150	150 x 150 mm	5	24	
ESPG100200	100 x 200 mm	5	24	
ESPG100250	100 x 250 mm	5	24	
ESPG100300	100 x 300 mm	5	24	
ESPG125125	125 x 125 mm	5	24	
ESPG130200	130 x 200 mm	5	24	
ESPG175175	175 x 175 mm	3	24	
ESPG150200	150 x 200 mm	3	24	
ESPG200200	200 x 200 mm	3	24	
ESPG200300	200 x 300 mm	3	24	
ESPG6080	60 x 80 mm Tracheostomy	10	48	
ESPG6565	65 x 65 mm Tracheostomy	10	48	
ESPG8080	80 x 80 mm Tracheostomy	10	48	
ESPG8090	80 x 90 mm Tracheostomy	10	48	
ESPG165180	165 x 180 mm Concave	3	24	
ESPG170170	170 x 170 mm Small Sacral	3	24	
ESPG220220	220 x 220 mm Sacral	3	24	
ESPG220220N	220 x 220 mm	3	24	
ESPG5050	50x50 mm	10	48	
ESPG20300	20x300 mm	5	24	
ESPG100120	100x120 mm	5	24	
ESPG100150	100 x 150 mm	5	24	
ESPG100350	100x350 mm	5	24	
ESPG200400	200x400 mm	3	24	
ESPG50	50Ø mm	10	48	
ESPG100	100Ø mm	10	48	
ESPG120130	120x130 mm heel	3	24	
ESPG100125	100x125 mm	5	24	
ESPG200250	200x250 mm	3	24	
ESPG6060	60x60mm Tracheostomy	10	48	
ESPG5060	50x60mm	10	48	
ESPG200500	200X500mm	3	24	
ESPG50125	50x125mm	5	24	
ESPG130150	130x150mm	5	24	
ESPG175175H	175x175mm heel	3	24	
ESPG5060H	50x60mm heel	10	48	
ESPG5070	50x70mm	10	48	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Lite	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGL7575	75 x 75 mm	10	48	
ESPGL75100	75 x 100 mm	10	48	
ESPGL100100	100 x 100 mm	10	48	
ESPGL150150	150 x 150 mm	5	24	
ESPGL100200	100 x 200 mm	5	24	
ESPGL100250	100 x 250 mm	5	24	
ESPGL100300	100 x 300 mm	5	24	
ESPGL125125	125 x 125 mm	5	24	
ESPGL130200	130 x 200 mm	5	24	
ESPGL175175	175 x 175 mm	3	24	
ESPGL150200	150 x 200 mm	3	24	
ESPGL200200	200 x 200 mm	3	24	
ESPGL200300	200 x 300 mm	3	24	
ESPGL200500	200 x 500 mm	3	24	
ESPGL6080	60 x 80 mm Tracheostomy	10	48	
ESPGL6565	65 x 65 mm Tracheostomy	10	48	
ESPGL8080	80 x 80 mm Tracheostomy	10	48	
ESPGL8090	80 x 90 mm Tracheostomy	10	48	
ESPGL165180	165 x 180 mm Concave	3	24	
ESPGL170170	170 x 170 mm Small Sacral	3	24	
ESPGL220220	220 x 220 mm Sacral	3	24	
ESPGL220220N	220 x 220 mm	3	24	
ESPGL5050	50x50 mm	10	48	
ESPGL20300	20x300 mm	5	24	
ESPGL100120	100x120 mm	5	24	
ESPGL100150	100 x 150 mm	5	24	
ESPGL100350	100x350 mm	5	24	
ESPGL200400	200x400 mm	3	24	
ESPGL50	50Ø mm	10	48	
ESPGL100	100Ø mm	10	48	
ESPGL120130	120x130 mm heel	3	24	
ESPGL100125	100x125 mm	5	24	
ESPGL200250	200x250 mm	3	24	
ESPGL6060	60x60 mm Tracheostomy	10	48	
ESPGL50125	50x125mm	5	24	
ESPGL130150	130x150mm	5	24	
ESPGL175175H	175x175mm heel	3	24	
ESPGL5060H	50x60mm heel	10	48	
ESPGL5060	50x60mm	10	48	
ESPGL5070	50x70mm	10	48	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Transfer	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGT7575	75 x 75 mm	10	48	
ESPGT75100	75 x 100 mm	10	48	
ESPGT100100	100 x 100 mm	10	48	
ESPGT150150	150 x 150 mm	5	24	
ESPGT100200	100 x 200 mm	5	24	
ESPGT100250	100 x 250 mm	5	24	
ESPGT100300	100 x 300 mm	5	24	
ESPGT125125	125 x 125 mm	5	24	
ESPGT130200	130 x 200 mm	5	24	
ESPGT175175	175 x 175 mm	3	24	
ESPGT150200	150 x 200 mm	3	24	
ESPGT200200	200 x 200 mm	3	24	
ESPGT6080	60 x 80 mm Tracheostomy	10	48	
ESPGT6565	65 x 65 mm Tracheostomy	10	48	
ESPGT8080	80 x 80 mm Tracheostomy	10	48	
ESPGT8090	80 x 90 mm Tracheostomy	10	48	
ESPGT165180	165 x 180 mm Concave	3	24	
ESPGT170170	170 x 170 mm Small Sacral	3	24	
ESPGT220220	220 x 220 mm Sacral	3	24	
ESPGT220220N	220 x 220 mm	3	24	
ESPGT5050	50x50 mm	10	48	
ESPGT20300	20x300 mm	5	24	
ESPGT100120	100x120 mm	5	24	
ESPGT100150	100 x 150 mm	5	24	
ESPGT100350	100x350 mm	5	24	
ESPGT50	50Ø mm	10	48	
ESPGT100	100Ø mm	10	48	
ESPGT100125	100x125 mm	5	24	
ESPGT6060	60x60 mm Tracheostomy	10	48	
ESPGT200500	200x500 mm	3	24	
ESPGT200250	200x250 mm	3	24	
ESPGT200300	200x300 mm	3	24	
ESPGT200400	200x400 mm	3	24	
ESPGT120130	120x130 mm heel	5	24	
ESPGT50125	50x125mm	5	24	
ESPGT130150	130x150mm	5	24	
ESPGT175175H	175x175mm heel	3	24	
ESPGT5060H	50x60mm heel	10	48	
ESPGT5060	50x60mm	10	48	
ESPGT5070	50x70mm	10	48	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Xtra Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGCEX7575	75 x 75 mm	10	48	
ESPGCEX75100	75 x 100 mm	10	48	
ESPGCEX100100	100 x 100 mm	10	48	
ESPGCEX100150	100 x 150 mm	5	24	
ESPGCEX100200	100 x 200 mm	5	24	
ESPGCEX100250	100 x 250 mm	5	24	
ESPGCEX100300	100 x 300 mm	5	24	
ESPGCEX125125	125 x 125 mm	5	24	
ESPGCEX150150	150 x 150 mm	5	24	
ESPGCEX150200	150 x 200 mm	3	24	
ESPGCEX200200	200 x 200 mm	3	24	
ESPGCEX200300	200 x 300 mm	3	24	
ESPGCEX165180	165 x 180 mm Concave	3	24	
ESPGCEX170170	170 x 170 mm Small Sacral	3	24	
ESPGCEX220220	220 x 220 mm Sacral	3	24	
ESPGCEX220220N	220 x 220 mm	3	24	
ESPGCEX5050	50x50 mm	10	48	
ESPGCEX20300	20x300 mm	5	24	
ESPGCEX100120	100x120 mm	5	24	
ESPGCEX100350	100x350 mm	5	24	
ESPGCEX200400	200x400 mm	3	24	
ESPGCEX100125	100x125 mm	5	24	
ESPGCEX5070	50x70mm	10	48	
ESPGCEX50125	50x125mm	5	24	
ESPGCEX130150	130x150mm	5	24	
ESPGCEX200500	200x500mm	3	24	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Lite Xtra Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGCEXL7575	75 x 75 mm	10	48	
ESPGCEXL75100	75 x 100 mm	10	48	
ESPGCEXL100100	100 x 100 mm	10	48	
ESPGCEXL100150	100 x 150 mm	5	24	
ESPGCEXL100200	100 x 200 mm	5	24	
ESPGCEXL100250	100 x 250 mm	5	24	
ESPGCEXL100300	100 x 300 mm	5	24	
ESPGCEXL125125	125 x 125 mm	5	24	
ESPGCEXL150150	150 x 150 mm	5	24	
ESPGCEXL150200	150 x 200 mm	3	24	
ESPGCEXL200200	200 x 200 mm	3	24	
ESPGCEXL200300	200 x 300 mm	3	24	
ESPGCEXL165180	165 x 180 mm Concave	3	24	
ESPGCEXL170170	170 x 170 mm Small Sacral	3	24	
ESPGCEXL220220	220 x 220 mm Sacral	3	24	
ESPGCEXL220220N	220 x 220 mm	3	24	
ESPGCEXL5050	50x50 mm	10	48	
ESPGCEXL20300	20x300 mm	5	24	
ESPGCEXL100120	100x120 mm	5	24	
ESPGCEXL100350	100x350 mm	5	24	
ESPGCEXL200400	200x400 mm	3	24	
ESPGCEXL100125	100x125 mm	5	24	
ESPGCEXL200500	200x500mm	3	24	
ESPGCEXL5070	50x70mm	10	48	
ESPGCEXL50125	50x125mm	5	24	
ESPGCEXL130150	130x150mm	5	24	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGC7575	75 x 75 mm	10	48	
ESPGC75100	75 x 100 mm	10	48	
ESPGC100100	100 x 100 mm	10	48	
ESPGC100150	100 x 150 mm	5	24	
ESPGC100200	100 x 200 mm	5	24	
ESPGC100250	100 x 250 mm	5	24	
ESPGC100300	100 x 300 mm	5	24	
ESPGC125125	125 x 125 mm	5	24	
ESPGC150150	150 x 150 mm	5	24	
ESPGC150200	150 x 200 mm	3	24	
ESPGC200200	200 x 200 mm	3	24	
ESPGC200300	200 x 300 mm	3	24	
ESPGC165180	165 x 180 mm Concave	3	24	
ESPGC170170	170 x 170 mm Small Sacral	3	24	
ESPGC220220	220 x 220 mm Sacral	3	24	
ESPGC220220N	220 x 220 mm	3	24	
ESPGC5050	50x50 mm	10	48	
ESPGC20300	20x300 mm	5	24	
ESPGC100120	100x120 mm	5	24	
ESPGC100350	100x350 mm	5	24	
ESPGC200400	200x400 mm	3	24	
ESPGC100125	100x125 mm	5	24	
ESPGC175200	175x200mm	3	24	
ESPGC250300	250x300mm	3	24	
ESPGC150175	150x175 mm	3	24	
ESPGC175175	175x175 mm	3	24	
ESPGC200500	200x500mm	3	24	
ESPGC170170N	170x170mm	3	24	
ESPGC5070	50x70mm	10	48	
ESPGC50125	50x125mm	5	24	
ESPGC130150	130x150mm	5	24	
ESPGC175175H	175x175mm heel	3	24	
ESPGC220250	220x250mm sacral	3	24	
ESPGC250250	250x250mm sacral	3	24	
ESPGC225225H	225x225mm heel	3	24	
ESPGC220250H	220x250mm heel	3	24	
ESPGC250250H	250x250mm heel	3	24	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Lite Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGCL7575	75 x 75 mm	10	48	
ESPGCL75100	75 x 100 mm	10	48	
ESPGCL100100	100 x 100 mm	10	48	
ESPGCL100150	100 x 150 mm	5	24	
ESPGCL100200	100 x 200 mm	5	24	
ESPGCL100250	100 x 250 mm	5	24	
ESPGCL100300	100 x 300 mm	5	24	
ESPGCL125125	125 x 125 mm	5	24	
ESPGCL150150	150 x 150 mm	5	24	
ESPGCL150200	150 x 200 mm	3	24	
ESPGCL200200	200 x 200 mm	3	24	
ESPGCL165180	165 x 180 mm Concave	3	24	
ESPGCL170170	170 x 170 mm Small Sacral	3	24	
ESPGCL220220	220 x 220 mm Sacral	3	24	
ESPGCL220220N	220 x 220 mm	3	24	
ESPGCL5050	50x50 mm	10	48	
ESPGCL100120	100x120 mm	5	24	
ESPGCL100350	100x350 mm	5	24	
ESPGCL200400	200x400 mm	3	24	
ESPGCL100125	100x125 mm	5	24	
ESPGCL175200	175x200mm	3	24	
ESPGCL250300	250x300mm	3	24	
ESPGCL150175	150x175 mm	3	24	
ESPGCL175175	175x175 mm	3	24	
ESPGCL200300	200x300mm	3	24	
ESPGCL200500	200x500mm	3	24	
ESPGCL170170N	170x170mm	3	24	
ESPGCL20300	20x300mm	5	24	
ESPGCL5070	50x70mm	10	48	
ESPGCL50125	50x125mm	5	24	
ESPGCL130150	130x150mm	5	24	
ESPGCL175175H	175x175mm heel	3	24	
ESPGCL220250	220x250mm sacral	3	24	
ESPGCL250250	250x250mm sacral	3	24	
ESPGCL225225H	225x225mm heel	3	24	
ESPGCL220250H	220x250mm heel	3	24	
ESPGCL250250H	250x250mm heel	3	24	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle C Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGCC7575	75 x 75 mm	10	48	
ESPGCC75100	75 x 100 mm	10	48	
ESPGCC100100	100 x 100 mm	10	48	
ESPGCC100150	100 x 150 mm	5	24	
ESPGCC100200	100 x 200 mm	5	24	
ESPGCC100250	100 x 250 mm	5	24	
ESPGCC100300	100 x 300 mm	5	24	
ESPGCC125125	125 x 125 mm	5	24	
ESPGCC150150	150 x 150 mm	5	24	
ESPGCC150200	150 x 200 mm	3	24	
ESPGCC200200	200 x 200 mm	3	24	
ESPGCC200300	200 x 300 mm	3	24	
ESPGCC200500	200 x 500 mm	3	24	
ESPGCC165180	165 x 180 mm Concave	3	24	
ESPGCC170170	170 x 170 mm Small Sacral	3	24	
ESPGCC220220	220 x 220 mm Sacral	3	24	
ESPGCC220220N	220 x 220 mm	3	24	
ESPGCC5050	50x50 mm	10	48	
ESPGCC20300	20x300 mm	5	24	
ESPGCC100120	100x120 mm	5	24	
ESPGCC100350	100x350 mm	5	24	
ESPGCC200400	200x400 mm	3	24	
ESPGCC100125	100x125 mm	5	24	
ESPGCC175200	175x200mm	3	24	
ESPGCC250300	250x300mm	3	24	
ESPGCC150175	150x175 mm	3	24	
ESPGCC17575	175x175 mm	3	24	
ESPGCC170170N	170x170mm	3	24	
ESPGCC5070	50x70mm	10	48	
ESPGCC50125	50x125mm	5	24	
ESPGCC130150	130x150mm	5	24	
ESPGCC175175H	175x175mm heel	3	24	
ESPGCC220250	220x250mm sacral	3	24	
ESPGCC250250	250x250mm sacral	3	24	
ESPGCC225225H	225x225mm heel	3	24	
ESPGCC220250H	220x250mm heel	3	24	
ESPGCC250250H	250x250mm heel	3	24	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Lite C Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGCCL7575	75 x 75 mm	10	48	
ESPGCCL75100	75 x 100 mm	10	48	
ESPGCCL100100	100 x 100 mm	10	48	
ESPGCCL100150	100 x 150 mm	5	24	
ESPGCCL100200	100 x 200 mm	5	24	
ESPGCCL100250	100 x 250 mm	5	24	
ESPGCCL100300	100 x 300 mm	5	24	
ESPGCCL125125	125 x 125 mm	5	24	
ESPGCCL150150	150 x 150 mm	5	24	
ESPGCCL150200	150 x 200 mm	3	24	
ESPGCCL200200	200 x 200 mm	3	24	
ESPGCCL200300	200 x 300 mm	3	24	
ESPGCCL200500	200 x 500 mm	3	24	
ESPGCCL165180	165 x 180 mm Concave	3	24	
ESPGCCL170170	170 x 170 mm Small Sacral	3	24	
ESPGCCL220220	220 x 220 mm Sacral	3	24	
ESPGCCL220220N	220 x 220 mm	3	24	
ESPGCCL5050	50x50 mm	10	48	
ESPGCCL20300	20x300 mm	5	24	
ESPGCCL100120	100x120 mm	5	24	
ESPGCCL100350	100x350 mm	5	24	
ESPGCCL200400	200x400 mm	3	24	
ESPGCCL100125	100x125 mm	5	24	
ESPGCCL175200	175x200mm	3	24	
ESPGCCL250300	250x300mm	3	24	
ESPGCCL150175	150x175 mm	3	24	
ESPGCCL175175	175x175 mm	3	24	
ESPGCCL170170N	170x170mm	3	24	
ESPGCCL5070	50x70mm	10	48	
ESPGCCL50125	50x125mm	5	24	
ESPGCCL130150	130x150mm	5	24	
ESPGCCL175175H	175x175mm heel	3	24	
ESPGCCL220250	220x250mm sacral	3	24	
ESPGCCL250250	250x250mm sacral	3	24	
ESPGCCL225225H	225x225mm heel	3	24	
ESPGCCL220250H	220x250mm heel	3	24	
ESPGCCL250250H	250x250mm heel	3	24	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle C Xtra Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGCCEX7575	75 x 75 mm	10	48	
ESPGCCEX75100	75 x 100 mm	10	48	
ESPGCCEX100100	100 x 100 mm	10	48	
ESPGCCEX100150	100 x 150 mm	5	24	
ESPGCCEX100200	100 x 200 mm	5	24	
ESPGCCEX100250	100 x 250 mm	5	24	
ESPGCCEX100300	100 x 300 mm	5	24	
ESPGCCEX125125	125 x 125 mm	5	24	
ESPGCCEX150150	150 x 150 mm	5	24	
ESPGCCEX150200	150 x 200 mm	3	24	
ESPGCCEX200200	200 x 200 mm	3	24	
ESPGCCEX200300	200 x 300 mm	3	24	
ESPGCCEX200500	200 x 500 mm	3	24	
ESPGCCEX165180	165 x 180 mm Concave	3	24	
ESPGCCEX170170	170 x 170 mm Small Sacral	3	24	
ESPGCCEX220220	220 x 220 mm Sacral	3	24	
ESPGCCEX220220N	220 x 220 mm	3	24	
ESPGCCEX5050	50x50 mm	10	48	
ESPGCCEX20300	20x300 mm	5	24	
ESPGCCEX100120	100x120 mm	5	24	
ESPGCCEX100350	100x350 mm	5	24	
ESPGCCEX200400	200x400 mm	3	24	
ESPGCCEX100125	100x125 mm	5	24	
ESPGCCEX5070	50x70mm	10	48	
ESPGCCEX50125	50x125mm	5	24	
ESPGCCEX130150	130x150mm	5	24	

ESPUMA® Gentle Family

Soft Silicone Polyurethane Foam Dressings

ESPUMA® Gentle Carbon Lite Xtra Comfort	Size	Pieces/Box	Boxes/Case	Case Dimensions
ESPGCCEXL7575	75 x 75 mm	10	48	
ESPGCCEXL75100	75 x 100 mm	10	48	
ESPGCCEXL100100	100 x 100 mm	10	48	
ESPGCCEXL100150	100 x 150 mm	5	24	
ESPGCCEXL100200	100 x 200 mm	5	24	
ESPGCCEXL100250	100 x 250 mm	5	24	
ESPGCCEXL100300	100 x 300 mm	5	24	
ESPGCCEXL125125	125 x 125 mm	5	24	
ESPGCCEXL150150	150 x 150 mm	5	24	
ESPGCCEXL150200	150 x 200 mm	3	24	
ESPGCCEXL200200	200 x 200 mm	3	24	
ESPGCCEXL200300	200 x 300 mm	3	24	
ESPGCCEXL200500	200 x 500 mm	3	24	
ESPGCCEXL165180	165 x 180 mm Concave	3	24	
ESPGCCEXL170170	170 x 170 mm Small Sacral	3	24	
ESPGCCEXL220220	220 x 220 mm Sacral	3	24	
ESPGCCEXL220220N	220 x 220 mm	3	24	
ESPGCCEXL5050	50x50 mm	10	48	
ESPGCCEXL20300	20x300 mm	5	24	
ESPGCCEXL100120	100x120 mm	5	24	
ESPGCCEXL100350	100x350 mm	5	24	
ESPGCCEXL200400	200x400 mm	3	24	
ESPGCCEXL100125	100x125 mm	5	24	
ESPGCCEXL5070	50x70mm	10	48	
ESPGCCEXL50125	50x125mm	5	24	
ESPGCCEXL130150	130x150mm	5	24	

- Amria - Alexandria - Egypt
 Tel. : 002 03 4500264 Fax : 002 03 4500263
 - Kafr-El-Zayat P.O.Box 38 Egypt
 Tel. : 002 040 2545643 Fax : 002 040 2545645
 E-mail : sales@pharmaplast-online.com



Pharmaplast Amria 23512, Alexandria - Egypt www.pharmaplast-online.com		Brochure artwork template no: I-TM /AR/001-6 (Version 3)						
Customer : Pharmaplast		Product name : Espuma® Gentle Family						
Issue date : 02-10-2012		Prepared by : Marina Medhat						
Colors :		Dimension : A4						
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					
2	05-05-2019	New Text by Jostina Ramsis	File maintenance					
3	23-01-2021	new table of sizes	File maintenance					
4	27-01-2021	adding missed sizes and codes	File maintenance					
5	18-05-2021	Add layer diagrams	File maintenance					
6	04-01-2023	Adding new codes	to be aligned with new coding					