

Lipo-Colloid dressings



Description

Lipo-Colloid dressings are non-adherent wound contact layer that may be laminated to Non-Woven fluff or Polyurethane foam absorbent layer or only.

How does it work?

1. Paraffinic components have non-polar fatty nature, so they do not adhere with the polar proteinaceous exudates.
2. Hydrocolloid particles absorb exudates and turn to gel which provides optimum moist environment that promotes wound healing. Furthermore, it aids non-adherence due to synergistic action between gel and paraffinic components.
3. The polyester net allows the passage of exudates to be absorbed by the absorbent layer, so excess exudates do not have time to dry at the wound bed.

Features:

• **Healing in moist environment:**

Hydrocolloid particles gel upon absorption of wound exudates, creating moist environment that promotes wound healing.

• **Painless and atraumatic removal:**

Due to its fatty, yet non-greasy, nature, together with the gel formation property upon exudates contact, Lipo- colloid dressings' removal is painless and atraumatic for the wound.

• **Absorbent:**

When laminated to PU foam or non-woven absorbent layer , the dressing manages different level of exudates.

• **Instant absorption:**

Exudates easily pass to the absorbent layer through the lipo-colloid mesh openings.

• **Exudates retained even under pressure:**

The absorbent layer can keep the absorbed exudates even under compression; this prevents maceration and retains a humid surface environment to facilitate wound healing.

• **Highly conformable:**

The dressing is very soft and conformable, so it conforms to wound contours.

• **Promote fast healing by stimulating fibroblasts proliferation.**

Available variants:

- **Non adherent wound layer:**

Pharma-Jelly Net: is a non adherent wound contact layer.

Indications:


Pharma-Jelly Net is used as primary wound dressing for its non adherent effect and to provide moist environment for better wound healing. It's suitable for:

1. Low to moderate exuding acute wounds:

- Minor burns. Skin abrasions. Traumatic wounds. Skin grafts and donor sites.

2. Low to moderate exuding chronic wounds:

- Leg and pressure ulcers. Venous ulcers.

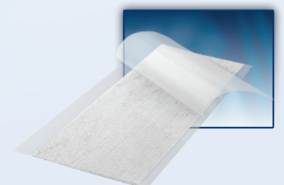

Lipo-Colloid dressings

II- For Minor wounds:

- 1- **Cure-Aid® Lipo:** Adhesive bandages for minor wounds.
- 2- **Cure-Aid® dressing strip Lipo:** Adhesive bandages for minor wounds, it could be cut to size.

Indications:

Used as primary dressings to cover minor wounds, to absorb any exudates and to provide moist environment in favour of the healing process.

III- For Low exuding wounds:

- 1- **Vilowond® Lipo:** absorbent non-adherent wound dressings composed of gelling non-adherent Lipo-colloid wound contact layer laminated to absorbent Non-woven fluff
- 2- **Pharmapore® Lipo:** soft and conformable post-operative non-woven dressing with **Vilowond® Lipo** central pad. The soft nonwoven touch is especially indicated for skin sensitive individuals.
- 3- **Pharmapore® PU Lipo** and **Pharmapore® PU Lipo Frame style** : Viral , bacterial and waterproof polyurethane film with **Vilowond® Lipo central pad**.
The frame style version is equipped with frame delivery system for easy application.
- 4- **Pharmapore® Ultra Lipo** : Nonwoven dressing coated with breathable waterproof, viral and bacterial proof polyurethane film outermost layer and **Vilowond® Lipo central pad**. **Pharmapore® Ultra Lipo** has the advantages of both **Pharmapore® Lipo** in being suitable for sensitive skin and **Pharmapore® PU Lipo** in being: Viral, bacterial and waterproof.

Indications:

Used as a primary dressing for acute wounds (burns, skin abrasions, traumatic wounds...) and chronic wounds (leg ulcers, pressure ulcers...).

- 5- **Pharmapore® IV Lipo:** soft and conformable cannula fixation non-woven dressing with **Vilowond® Lipo central pad**. The soft nonwoven touch is especially indicated for skin sensitive individuals.

- 6- **Pharmapore® PU IV Lipo:** Viral , bacterial and waterproof cannula fixation polyurethane film with **Vilowond® Lipo central pad**.

Indications:

For safe and secured cannula fixation.

- For Moderate to Highly exuding wounds:





- 1- **ESPUMA® Lipo** : absorbent non-adherent wound dressings composed of gelling non-adherent Lipo-colloid wound contact layer laminated to highly absorbent polyurethane foam
 - 2- **Espuma C lipo:** same as **Espuma Lipo** but contains odor adsorptive carbon layer.
 - 3- **Espuma Xtra lipo** same as **Espuma Lipo** but contains highly absorbent eXsorb layer .
- ESPUMA® Comfort Lipo** : in which **ESPUMA® Lipo** is placed as an island pad on a breathable, waterproof, bacteria proof and viral proof polyurethane adhesive border to allow. fixation without secondary dressing or fixation tape
- Espuma C comfort lipo:** same as **Espuma comfort Lipo** but contains odor adsorptive carbon layer
- Espuma Xtra comfort lipo** same as **Espuma comfort Lipo** but contains highly absorbent eXsorb layer

Indications:

ESPUMA® Lipo and **ESPUMA® Comfort Lipo** families are used as a primary dressing for chronic wounds (leg ulcers, pressure ulcers...) and acute wounds (burns, skin abrasions, traumatic wounds...).

Contraindications/Precautions:

- 1- Don't use in case of sensitivity to any of the product ingredients.
 - 2- The product is sterile, don't use if the pouch is damaged.
 - 3- The product is non-occlusive, so it could be used over infected wounds under medical supervision, and the dressing must be changed daily.
- The product must be stored flat, protected from heat, moisture and direct light

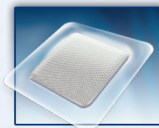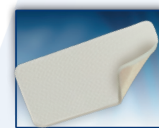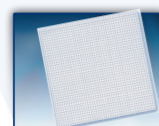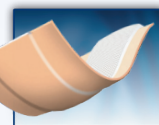

Lipo-Colloid dressings

Table of sizes:

ESPUMA [®] Lipo	Size	Pieces/ box	Boxes / Case
ESLIPO7575	75 x 75mm	10	48
ESLIPO75100	75 x 100mm	10	48
ESLIPO100100	100 x 100mm	10	48
ESLIPO100150	100 x 150mm	5	24
ESLIPO100200	100 x 200mm	5	24
ESLIPO150150	150 x 150 mm	5	24
ESLIPO150200	150 x 200mm	3	24
ESLIPO200200	200 x 200 mm	3	24
ESLIPO200300	200 x 300 Tracheostomy	5	24
ESLIPO8080	80 x 80 mm Tracheostomy	10	48
ESLIPO6060	60x60 mm Tracheostomy	10	48
ESLIPO6080	60x80 mm Tracheostomy	10	48
ESLIPO8090	80x90 mm Tracheostomy	10	48
ESLIPO9090	90x90 mm Tracheostomy	10	48
	90x90 mm Cavity	10	48
ESLIPO2590	25x90 mm Cavity	5	24
ESLIPO20400	20x400 mm Cavity	5	24
ESLIPO20450	20x450 mm Cavity	5	24
ESLIPO25450	25x450 mm Cavity	5	24
ESLIPO50400	50x400 mm Cavity	10	48
ESLIPO25400	25x400 mm Cavity	5	24
ESLIPO165180	165 x 180 Concave	3	24
ESLIPO220220	220 x 220 Sacral	3	24
ESLIPO100130	100 x 130 Heel	3	48
ESLIPO5050	50x50 mm Circular	10	48
ESLIPO100125	100x125 mm	5	24
ESLIPO125125	125x125 mm	5	24
ESLIPO175175	175x175 mm	5	24
ESLIPO150250	150x250 mm	3	24
ESLIPO200250	200x250 mm	3	24
ESLIPO300300	300x300 mm	3	24
ESLIPO400400	400x400 mm	3	24
ESLIPO100250	100 x 250 mm	5	24
ESLIPO100300	100 x 300 mm	3	24
ESLIPO130200	130 x 200 mm	3	24
ESLIPO200500	200 x 500 mm	3	24
ESLIPO170170	170 x 170 mm Sacral	3	24
ESLIPO20300	20x300 mm	5	24
ESLIPO100120	100x120 mm	5	24
ESLIPO100350	100x350 mm	3	24
ESLIPO200400	200x400 mm	3	24
ESLIPO50	50Ø mm	10	48
ESLIPO100	100Ø mm	10	48
ESLIPO6565	65x65 mm	10	48
ESLIPO170170N	170x170 mm		
ESLIPO220220N	220x220 mm		

Lipo-Colloid dressings

Table of sizes:

ESPUMA® Lipo Comfort	Size	Pieces/ box	Boxes / Case
ESPADHLIPO75100	75 x 100 mm	10	48
ESPADHLIPO100100	100 x 100 mm	10	48
ESPADHLIPO150150	150 x 150 mm	5	24
ESPADHLIPO150200	150 x 200 mm	3	24
ESPADHLIPO200200	200 x 200 mm	3	24
ESPADHLIPO165180	Concave 165 x 180 mm	3	24
ESPADHLIPO220220	Sacral 220 x 220 mm	3	24
ESPADHLIPO5050	50x50 mm	10	48
ESPADHLIPO6060	60x60 mm	10	48
ESPADHLIPO100125	100x125 mm	5	24
ESPADHLIPO125125	125x125mm	5	24
ESPADHLIPO175175	175x175 mm	5	24
ESPADHLIPO150250	150x250 mm	3	24
ESPADHLIPO200250	200x250 mm	3	24
ESPADHLIPO300300	300x300 mm	3	24
ESPADHLIPO400400	400x400 mm	3	24
ESPADHLIPO7575	75 x 75 mm	10	48
ESPADHLIPO100200	100 x 200 mm	5	24
ESPADHLIPO100250	100 x 250 mm	5	24
ESPADHLIPO100300	100 x 300 mm	3	24
ESPADHLIPO200300	200 x 300 mm	3	24
ESPADHLIPO200500	200 x 500 mm	3	24
ESPADHLIPO170170	170 x 170 mm Sacral	3	24
ESPADHLIPO100120	100 x 120 mm	5	24
ESPADHLIPO100150	100 x 150 mm	3	24
ESPADHLIPO100350	100 x 350 mm	3	24
ESPADHLIPO200400	200 x 400 mm	3	24
ESPADHLIPO50	50Ø mm	10	48
ESPADHLIPO100	100Ø mm	10	48
ESPADHLIPO120130	120 x 130 mm	3	24
ESPADHLIPO160180	160 x 180 mm	3	24
ESPADHLIPO170170N	170x170 mm		
ESPADHLIPO220220N	220x220 mm		
Vilowond® Lipo	Size	Pieces/ box	Boxes / Case
VILOLIPO5050	5x5 cm	10	48
VILOLIPO50100	5x10 cm	10	48
VILOLIPO50150	5x15 cm	10	48
VILOLIPO50200	5x20 cm	10	24
VILOLIPO75100	7.5x10 cm	10	48
VILOLIPO75150	7.5x15 cm	5	24
VILOLIPO75200	7.5x20 cm	5	24
VILOLIPO100100	10x10 cm	10	48
VILOLIPO100150	10x15 cm	5	24
VILOLIPO100200	10x20 cm	5	24
VILOLIPO100400	10x40 cm	5	24
VILOLIPO150200	15x20 cm	5	24

Lipo-Colloid dressings

Table of sizes:

Vilowond® Lipo	Size	Pieces/ box	Boxes / Case
VILOLIPO150150	15x15 cm	5	24
VILOLIPO150400	15x40 cm	5	24
VILOLIPO200200	20x20 cm	5	24
VILOLIPO200400	20x40 cm	5	24
VILOLIPO75400	7.5x40 cm	5	24
VILOLIPO105105	10.5x10.5 cm	5	24
VILOLIPO105195	10.5x19.5 cm	5	24
VILOLIPO751000	7.5 cmx1 m	1	24
VILOLIPO1001000	10 cmx 1 m	1	24
VILOLIPO1501000	15 cmx 1 m	1	24
VILOLIPO752000	7.5 cmx2 m	1	24
VILOLIPO1002000	10 cmx 2 m	1	24
VILOLIPO1502000	15 cmx 2 m	1	24
VILOLIPO757000	7.5 cmx7 m	1	24
VILOLIPO1007000	10 cmx 7 m	1	24
VILOLIPO1507000	15 cmx 7 m	1	24
VILOLIPO75500	7.5x50 cm	1	24
VILOLIPO100250	10x25 cm	5	24
VILOLIPO100300	10x30 cm	5	24
VILOLIPO100120	10x12cm	5	24
VILOLIPO100350	10x35 cm	5	24
VILOLIPO200250	20x25 cm	5	24
VILOLIPO200300	20x30 cm	3	24
VILOLIPO200350	20x35 cm	5	24
VILOLIPO300300	30x30 cm	5	24
VILOLIPO5075	5x7.5 cm	10	48
VILOLIPO20300	2x30 cm	5	24
VILOLIPO20400	2x40 cm	5	24
VILOLIPO25450	2.5x45 cm	5	24
VILOLIPO50300	5x30 cm	5	24
VILOLIPO70120	7x12 cm	5	24
VILOLIPO75120	7.5x12 cm	5	24
Cure-Aid® Lipo Dressing Strip	Size	Pieces/ box	Boxes / Case
DERMILIPO60100	60x100mm	10	120
DERMILIPO80100	80x100mm	10	120
DERMILIPO601000	60x1000mm	1	120
DERMILIPO801000	80x1000mm	1	120
Pharmapore® Lipo	Size	Pieces/ box	Boxes / Case
PORELIPO5070	5x7cm	10	48
PORELIPO6070	6x7cm	10	48
PORELIPO6080	6x8cm	10	48
PORELIPO5090	5x9cm	10	48
PORELIPO6083	6x8.3cm	10	48
PORELIPO50100	5x10cm	10	48
PORELIPO6082	6x8.25cm	10	48
PORELIPO6085	6x8.5cm	10	48

Lipo-Colloid dressings

Table of sizes:

Pharmapore® Lipo	Size	Pieces/ box	Boxes / Case
PORELIPO7080	7x8cm	10	48
PORELIPO6580	6.5x8cm	10	48
PORELIPO7575	7.5x7.5cm	10	48
PORELIPO6090	6x9cm	10	48
PORELIPO60100	6x10cm	10	48
PORELIPO5082	5x8.25cm	10	48
PORELIPO75100	7.5x10cm	10	48
PORELIPO80100	8x10cm	10	48
PORELIPO82100	8.25x10cm	10	48
PORELIPO100100	10x10cm	10	48
PORELIPO70120	7x12cm	10	48
PORELIPO80120	8x12cm	10	48
PORELIPO100120	10x12cm	10	48
PORELIPO83120	8.3x12cm	10	48
PORELIPO90150	9x15cm	10	48
PORELIPO80150	8x15cm	10	48
PORELIPO82150	8.25x15cm	10	48
PORELIPO100140	10x14cm	10	48
PORELIPO100150	10x15cm	10	48
PORELIPO83180	8.3x18cm	10	48
PORELIPO82200	8.25x20cm	10	24
PORELIPO90200	9x20cm	10	24
PORELIPO100200	10x20cm	10	24
PORELIPO83240	8.3x24cm	10	24
PORELIPO82250	8.25x25cm	10	24
PORELIPO90250	9x25cm	10	24
PORELIPO100250	10x25cm	10	24
PORELIPO82300	8.25x30cm	10	24
PORELIPO90300	9x30cm	10	24
PORELIPO100300	10x30cm	10	24
PORELIPO90350	9x35cm	10	24
PORELIPO82350	8.25x35cm	10	24
PORELIPO100350	10x35cm	10	24
PORELIPO83415	8.3x41.5cm	10	24
PORELIPO100160	10x16cm	10	48
PORELIPO100210	10x21cm	10	24
PORELIPO100260	10x26cm	10	24
PORELIPO100400	10x40cm	10	24
PORELIPO120160	12x16cm	10	48
PORELIPO150150	15x15cm	10	24
PORELIPO200200	20x20cm	10	24
PORELIPO2572	2.5x7.2cm	400	8
PORELIPO38100	3.8x10cm	10	48
PORELIPO5072	5x7.2cm	10	48
PORELIPO5075	5x7.5cm	10	48
PORELIPO70100	7x10cm	10	48
PORELIPO6075	6x7.5cm	10	48
PORELIPO85155	8.5x15.5cm	10	48

Lipo-Colloid dressings

Table of sizes:

Pharmapore® Lipo	Size	Pieces/ box	Boxes / Case
PORELIPO85150	8.5x15cm	10	48
PORELIPO8595	8.5x9.5cm	10	48
PORELIPO90100	9x10cm	10	48
PORELIPO150200	15x20cm		
Pharmapore® Ultra Lipo	Size	Pieces/ box	Boxes / Case
POREULIPO5070	5x7cm	10	48
POREULIPO6070	6x7cm	10	48
POREULIPO6080	6x8cm	10	48
POREULIPO5090	5x9cm	10	48
POREULIPO6083	6x8.3cm	10	48
POREULIPO50100	5x10cm	10	48
POREULIPO6082	6x8.25cm	10	48
POREULIPO6085	6x8.5cm	10	48
POREULIPO7080	7x8cm	10	48
POREULIPO6580	6.5x8cm	10	48
POREULIPO7575	7.5x7.5cm	10	48
POREULIPO6090	6x9cm	10	48
POREULIPO60100	6x10cm	10	48
POREULIPO5082	5x8.25cm	10	48
POREULIPO75100	7.5x10cm	10	48
POREULIPO80100	8x10cm	10	48
POREULIPO82100	8.25x10cm	10	48
POREULIPO100100	10x10cm	10	48
POREULIPO70120	7x12cm	10	48
POREULIPO80120	8x12cm	10	48
POREULIPO100120	10x12cm	10	48
POREULIPO83120	8.3x12cm	10	48
POREULIPO90150	9x15cm	10	48
POREULIPO80150	8x15cm	10	48
POREULIPO82150	8.25x15cm	10	48
POREULIPO100140	10x14cm	10	48
POREULIPO100150	10x15cm	10	48
POREULIPO83180	8.3x18cm	10	48
POREULIPO82200	8.25x20cm	10	24
POREULIPO90200	9x20cm	10	24
POREULIPO100200	10x20cm	10	24
POREULIPO83240	8.3x24cm	10	24
POREULIPO82250	8.25x25cm	10	24
POREULIPO90250	9x25cm	10	24
POREULIPO100250	10x25cm	10	24
POREULIPO82300	8.25x30cm	10	24
POREULIPO90300	9x30cm	10	24
POREULIPO100300	10x30cm	10	24
POREULIPO90350	9x35cm	10	24
POREULIPO82350	8.25x35cm	10	24
POREULIPO100350	10x35cm	10	24
POREULIPO83415	8.3x41.5cm	10	24
POREULIPO100160	10x16 cm	10	48

Lipo-Colloid dressings

Table of sizes:

Pharmapore® Ultra Lipo		Size	Pieces/ box	Boxes / Case
POREULIPO150150		15x15 cm	10	24
POREULIPO200200		20x20 cm	10	24
POREULIPO5072		5x7.2 cm	10	48
POREULIPO5075		5x7.5 cm	10	48
POREULIPO70100		7x10 cm	10	48
POREULIPO6075		6x7.5 cm	10	48
POREULIPO85155		8.5x15.5cm	10	48
POREULIPO85150		8.5x15cm	10	48
POREULIPO8595		8.5x9.5cm	10	48
POREULIPO90100		9x10cm	10	48
POREULIP150200		15x20cm		
Pharmapore® PU Lipo	Pharmapore® PU FS Lipo	Size	Pieces/ box	Boxes / Case
POREPULIPO5070	POREPUFSLIPO5070	5x7cm	10	48
POREPULIPO6070	POREPUFSLIPO6070	6x7cm	10	48
POREPULIPO6080	POREPUFSLIPO6080	6x8cm	10	48
POREPULIPO5090	POREPUFSLIPO5090	5x9cm	10	48
POREPULIPO6083	POREPUFSLIPO6083	6x8.3cm	10	48
POREPULIPO50100	POREPUFSLIPO50100	5x10cm	10	48
POREPULIPO6082	POREPUFSLIPO6082	6x8.25cm	10	48
POREPULIPO6085	POREPUFSLIPO6085	6x8.5cm	10	48
POREPULIPO7080	POREPUFSLIPO7080	7x8cm	10	48
POREPULIPO6580	POREPUFSLIPO6580	6.5x8cm	10	48
POREPULIPO7575	POREPUFSLIPO7575	7.5x7.5cm	10	48
POREPULIPO6090	POREPUFSLIPO6090	6x9cm	10	48
POREPULIPO60100	POREPUFSLIPO60100	6x10cm	10	48
POREPULIPO5082	POREPUFSLIPO5082	5x8.25cm	10	48
POREPULIPO75100	POREPUFSLIPO75100	7.5x10cm	10	48
POREPULIPO80100	POREPUFSLIPO80100	8x10cm	10	48
POREPULIPO82100	POREPUFSLIPO82100	8.25x10cm	10	48
POREPULIPO100100	POREPUFSLIPO100100	10x10cm	10	48
POREPULIPO70120	POREPUFSLIPO70120	7x12cm	10	48
POREPULIPO80120	POREPUFSLIPO80120	8x12cm	10	48
POREPULIPO100120	POREPUFSLIPO100120	10x12cm	10	48
POREPULIPO83120	POREPUFSLIPO83120	8.3x12cm	10	48
POREPULIPO90150	POREPUFSLIPO90150	9x15cm	10	48
POREPULIPO80150	POREPUFSLIPO80150	8x15cm	10	48
POREPULIPO82150	POREPUFSLIPO82150	8.25x15cm	10	48
POREPULIPO100140	POREPUFSLIPO100140	10x14cm	10	48
POREPULIPO100150	POREPUFSLIPO100150	10x15cm	10	48
POREPULIPO83180	POREPUFSLIPO83180	8.3x18cm	10	48
POREPULIPO82200	POREPUFSLIPO82200	8.25x20cm	10	24
POREPULIPO90200	POREPUFSLIPO90200	9x20cm	10	24
POREPULIPO100200	POREPUFSLIPO100200	10x20cm	10	24
POREPULIPO83240	POREPUFSLIPO83240	8.3x24cm	10	24
POREPULIPO82250	POREPUFSLIPO82250	8.25x25cm	10	24
POREPULIPO90250	POREPUFSLIPO90250	9x25cm	10	24
POREPULIPO100250	POREPUFSLIPO100250	10x25cm	10	24

Lipo-Colloid dressings

Table of sizes:

Pharmapore® PU Lipo	Pharmapore® PU FS Lipo	Size	Pieces/ box	Boxes / Case
POREPULIPO82300	POREPUFSLIPO82300	8.25x30cm	10	24
POREPULIPO90300	POREPUFSLIPO90300	9x30cm	10	24
POREPULIPO100300	POREPUFSLIPO100300	10x30cm	10	24
POREPULIPO90350	POREPUFSLIPO90350	9x35cm	10	24
POREPULIPO82350	POREPUFSLIPO82350	8.25x35cm	10	24
POREPULIPO100350	POREPUFSLIPO100350	10x35cm	10	24
POREPULIPO83415	POREPUFSLIPO83415	8.3x41.5cm	10	24
POREPULIPO100160	POREPUFSLIPO100160	10x16cm	10	48
POREPULIPO100210	POREPUFSLIPO100210	10x21cm	10	24
POREPULIPO100260	POREPUFSLIPO100260	10x26cm	10	24
POREPULIPO100400	POREPUFSLIPO100400	10x40cm	10	24
POREPULIPO120160	POREPUFSLIPO120160	12x16cm	10	48
POREPULIPO150150	POREPUFSLIPO150150	15x15cm	10	24
POREPULIPO200200	POREPUFSLIPO200200	20x20cm	10	24
POREPULIPO5072	POREPUFSLIPO5072	5x7.2cm	10	48
POREPULIPO5075	POREPUFSLIPO5075	5x7.5cm	10	48
POREPULIPO70100	POREPUFSLIPO70100	7x10cm	10	48
POREPULIPO6075	POREPUFSLIPO6075	6x7.5cm	10	48
POREPULIPO85155	POREPUFSLIPO85155	8.5x15.5cm	10	48
POREPULIPO85150	POREPUFSLIPO85150	8.5x15cm	10	48
POREPULIP8595	POREPUFSLIPO8595	8.5x9.5cm	10	48
POREPULIPO90100	POREPUFSLIPO90100	9x10cm	10	48
POREPULIPO150200	POREPUFSLIPO150200	15x20cm		
Pharmapore® IV Lipo	Pharmapore® PU IV Lipo	Size	Pieces/ box	Boxes / Case
IVNWLIPO6070	IVPULIPO6070	6X7CM	100	16
IVNWLIPO7070	IVPULIPO7070	7X7CM	100	16
IVNWLIPO7080	IVPULIPO7080	7X8CM	100	16
IVNWLIPO80100	IVPULIPO80100	8X10CM	100	16
IVNWLIPO82100	IVPULIPO82100	8.25X10CM	100	16
IVNWLIPO6085	IVPULIPO6085	6X8.5CM	100	16
IVNWLIPO7085	IVPULIPO7085	7X8.5CM	100	16
IVNWLIPO85100	IVPULIPO85100	8.5X10CM	100	16
IVNWLIPO7090	IVPULIPO7090	7X9CM	100	16
IVNWLIPO90100	IVPULIPO90100	9X10CM	100	16
IVNWLIPO100120	IVPULIPO100120	10X12CM	60	16
IVNWLIPO100140	IVPULIPO100140	10X14CM	60	16
IVNWLIPO6080	IVPULIPO6080	6X8CM	100	16
IVNWLIPO85115	IVPULIPO85115	8.5X11.5CM	60	16
IVNWLIPO5057	IVPULIPO5057t	5X5.7CM	100	16
IVNWLIPO100155		10X15.5CMt	60	16
Pharmapore® PU IV FS Lipo		Size	Pieces/ box	Boxes / Case
IVFSLIPO6070		6X7CM	100	16
IVFSLIPO7070		7X7CM	100	16
IVFSLIPO7080		7X8CM	100	16
IVFSLIPO80100		8X10CM	100	16
IVFSLIPO82100		8.25X10CM	100	16
IVFSLIPO6085		6X8.5CM	100	16

Lipo-Colloid dressings

Table of sizes:

Pharmapore® PU IV FS Lipo	Size	Pieces/ box	Boxes / Case
IVFSLIPO7085	7X8.5CM	100	16
IVFSLIPO85100	8.5X10CM	100	16
IVFSLIPO7090	7X9CM	100	16
IVFSLIPO90100	9X10CM	100	16
IVFSLIPO100120	10X12CM	60	16
IVFSLIPO100140	10X14CM	60	16
IVFSLIPO6080	6X8CM	100	16
IVFSLIPO85115	8.5X11.5CM	60	16
IVFSLIPO5057	5X5.7CM	100	16
IVFSLIPO100120ADV	10X12CM	50	24
IVFSLIPO100155ADV	10x15.5CM	50	24
IVFSLIPO6570ADV	6.5x7CM	50	24
IVFSLIPO7085ADV	7X8.5CM	50	24
IVFSLIPO85115ADV	8.5x11.5CM	50	24
Pharma-Jelly Net®	Size	Pieces/ box	Boxes / Case
JN5050	5 x 5 cm	10	48
JN50100	5 x 10 cm	10	48
JN100100	10 x 10 cm	10	48
		100	16
JN100150	10 x 15 cm	10	16
JN100200	10 x 20 cm	10	24
		50	16
JN100300	10 x 30 cm	10	24
JN150150	15 x 15 cm	10	24
JN150200	15 x 20 cm	10	24
JN200200	20 x 20 cm	10	24
JN200300	20 x 30 cm	10	24
JN120250	12 x 25 cm	10	24
JN150250	15 x 25 cm	10	24
JN150300	15 x 30 cm	10	24
JN7575	7.5 x 7.5 cm	10	48
JN75100	7.5 x 10 cm	10	48
JN75200	7.5 x 20 cm	10	24
JN75400	7.5 x 40 cm	10	24
JN751300	7.5 x 1.3 m	1	48
JN751000	7.5 cm x 1m	1	48
JN1001000	10 cm x 1m	1	48
JN1501000	15 cm x 1m	1	48
JN2001000	20 cm x 1 m	1	48
JN1002000	10 cm x 2 m	1	48
JN1502000	15 cm x 2 m	1	48
JN2002000	20 cm x 2 m	1	48
JN100400	10x40 cm	10	24
JN200400	20x40 cm	10	24
JN501000	5x100 cm	2	48
JN757000	7.5x700 cm	1	48
JN1007000	10x700 cm	1	48
JN1507000	15x700 cm	1	48
JN75150	7.5x15 cm	10	24
JN150400	15x40 cm	10	24

- Amria - Alexandria - Egypt
 Tel. : 002 03 4500264 Fax : 002 03 4500263
 - Kafr-El-Zayat P.O.Box 38 Egypt
 Tel. : 002 040 2545643 Fax : 002 040 2545645
 E-mail : sales@pharmaplast-online.com

 **Pharmaplast**
 www.pharmaplast-online.com

Pharmaplast Amria 23512, Alexandria - Egypt www.pharmaplast-online.com		Brochure artwork template no: I-TM /AR/001-6 (Version 3)						
Customer : Pharmaplast		Product name : Lipo-Colloid dressings						
Issue date : 16-11-2016		Prepared by : Marina Medhat						
Colors : CMYK		Dimension : A4						
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					
2	29-12-2018	New Text by Wassim Gad	Essential requirement					
3	05-06-2022	Updating codes	File maintenance					