

# Pharmagel Family



## Description:

**Pharmagel** is a sterile transparent polyurethane hydrogel dressing containing 60% water.

**Pharmagel** provides moisture to the wound bed and at the same time, it absorbs exudates locking it inside, this moisture balance promotes granulation and faster epithelialization.

**Pharmagel Comfort:** Virus-proof, bacteria-proof and waterproof adhesive polyurethane film with **Pharmagel** central pad, to allow fixation without secondary dressing or fixation tape.

**Pharmagel Breast Pads** is designed to provide pain relief for sore nipples.

**Cure-Aid® Hydrogel bandages:** adhesive hydrogel bandages.

## Mechanism of action:

When applied to the wound, **Pharmagel** provides moisture to the wound and at the same time, it absorbs exudates locking it inside. This moisture balance promotes granulation and faster epithelialization.

## Features:

- **Pharmagel** creates a moist environment, which:
  1. Stimulates wound healing, making **Pharmagel** suitable for slow-healing chronic wounds.
  2. Facilitates debridement of necrotic tissues.
  3. Accelerates epithelialization process with reduced scarring.
- **Pharmagel** products can be left on the wound for longer times due to its sustained absorption and transparency.
- **Pharmagel** products doesn't dissolve in absorbed exudates, thus it can be easily removed as one piece, leaving no residue on wound.
- **Pharmagel** products is non-adherent, facilitating painless atraumatic dressing removal.
- Due to its high water content, **Pharmagel** products has as a slight cooling and soothing effects which provide instant pain relieve and aid patient comfort.
- The dressing is breathable, yet it's bacteria- proof, virus-proof and waterproof.
- The dressing has cushioning effect protecting the skin against friction and pressure.
- Ideal solution for sore nipples:
  - Due to its cooling effect, **Pharmagel** is an ideal solution to soothe, relieve and help heal sore nipples.
  - The gel is absorbent, thus it helps prevent leakage.
  - The gel prevents friction with the bra due to its cushioning effect.

## Intended Purpose:

- 1- **Pharmagel** and **Pharmagel Comfort:**
  - Suitable for treatment of all wounds in granulation and epithelialisation phase with no to low exudate level.
  - Due to **Pharmagel**'s high water content, it is suitable for use in debridement.
- 2- **Pharmagel Breast Pads** are used to provide instant cooling relief from cracked, painful or sore nipples.
- 3- **Cure-Aid® Hydrogel bandages:** Protection and management of minor cuts, scrapes, bites, blisters, corns and burns.

# Pharmagel Family

| Pharmagel           | Size        | Pieces/Box | Boxes/Case | Case Dimensions |
|---------------------|-------------|------------|------------|-----------------|
| GEL100100           | 10cmx10cm   | 5          | 48         | 54x40x17        |
| GEL150150           | 15cmx15cm   | 5          | 24         | 43x37x25        |
| GEL200200           | 20cmx20cm   | 5          | 24         | 43x25x29        |
| GEL100200           | 10cmx20cm   | 5          | 24         | 43x25x29        |
| GEL150200           | 15cmx20cm   | 5          | 24         | 43x25x29        |
| GEL300600           | 30cmx60cm   | 5          | 24         |                 |
| GEL100150           | 10cmx15cm   | 5          | 24         | 43x37x25        |
| GEL4565             | 4.5cmx6.5cm | 5          | 48         | 54x40x17        |
| GEL7075             | 7cmx7.5cm   | 5          | 48         | 54x40x17        |
| GEL75100            | 7.5cmx10cm  | 5          | 48         |                 |
| GEL200300           | 20cmx30cm   | 5          | 24         | 43x25x29        |
| GEL125180           | 12.5cmx18cm | 5          | 24         | 54x40x17        |
| GEL5075             | 5cmx7.5cm   | 5          | 48         | 54x40x17        |
| GEL6060             | 6cmx6cm     | 5          | 48         | 43x25x29        |
| GEL85120            | 8.5cmx12cm  | 5          | 48         |                 |
| GEL60               | 60mmØ       | 5          | 48         |                 |
| Pharmagel Comfort   | Size        | Pieces/Box | Boxes/Case | Case Dimensions |
| GELC4565            | 45x65mm     | 5          | 48         | 54x40x17        |
| GELC75100           | 75x100mm    | 5          | 48         | 54x40x17        |
| GELC100100          | 100x100mm   | 5          | 48         | 54x40x17        |
| GELC150150          | 150x150mm   | 5          | 24         | 43x37x25        |
| GELC200200          | 200x200mm   | 5          | 24         | 43x25x29        |
| GELC67110           | 67x110mm    | 5          | 48         | 54x40x17        |
| GELC4368            | 43x68mm     | 5          | 48         | 54x40x17        |
| GELC2868            | 28x68mm     | 5          | 48         | 54x40x17        |
| GELC2235            | 22x35mm     | 5          | 48         | 54x40x17        |
| GELC2665            | 26x65mm     | 5          | 48         | 54x40x17        |
| GELC3070            | 30x70mm     | 5          | 48         | 54x40x17        |
| GELC125125          | 125x125mm   | 5          | 24         |                 |
| GELC60100           | 60x100mm    | 5          | 48         | 54x40x17        |
| GELC7575            | 75x75mm     | 5          | 48         | 54x40x17        |
| GELC/PET2045        | 20x45mm     | 6          | 48         | 54x40x17        |
| GELC/PET4368        | 43x68mm     | 6          | 48         | 54x40x17        |
| Pharmagel Face Mask | Size        | Pieces/Box | Boxes/Case | Case Dimensions |
| GEL3545             | 3.5cmx4.5cm | 1          | 24         |                 |
| GEL2535             | 2.5cmx3.5cm | 1          | 24         |                 |

# Pharmagel Family

| Pharmagel Blanket    | Size        | Pieces/Box | Boxes/Case | Case Dimensions |
|----------------------|-------------|------------|------------|-----------------|
| GEL500600            | 50cmx60cm   | 1          | 10         |                 |
| Pharmagel Breast Pad | Size        | Pieces/Box | Boxes/Case | Case Dimensions |
| GEL7075              | 7cmx7.5cm   | 5          | 48         | 26x25x30        |
| GEL60                | 6cmØ        | 5          | 48         | 26x25x30        |
| Cure-Aid® Hydrogel   | Size        | Pieces/Box | Boxes/Case | Case Dimensions |
| CUREGEL4565          | 4.5cmx6.5cm | 6          | 48         | 54x40x17        |
| CUREGEL4565ADH       | 4.5cmx6.5cm | 6          | 48         | 54x40x17        |
| CUREGEL4368PET       | 4.3cmx6.8cm | 6          | 48         | 54x40x17        |
| CUREGEL4368ADH       | 4.3cmx6.8cm | 6          | 48         | 54x40x17        |
| CUREGEL2868ADH       | 2.8cmx6.8cm | 6          | 48         | 54x40x17        |
| CUREGEL2045PET       | 2cmx4.5cm   | 6          | 48         | 54x40x17        |
| CUREGEL2045          | 2cmx45mm    | 6          | 48         | 54x40x17        |
| CUREGEL2665ADH       | 2.6cmx6.5cm | 6          | 48         | 54x40x17        |
| CUREGEL3070ADH       | 3cmx7cm     | 6          | 48         | 54x40x17        |
| CUREGEL60100ADH      | 6cmx10cm    | 6          | 48         | 54x40x17        |

- Amria - Alexandria - Egypt  
 Tel. : 002 03 4500264 Fax : 002 03 4500263  
 - Kafr-El-Zayat P.O.Box 38 Egypt  
 Tel. : 002 040 2545643 Fax : 002 040 2545645  
 E-mail : sales@pharmaplast-online.com



www.pharmaplast-online.com

| Issue Number |  | Revision Date | Description of Change  | Rationale For Change                     | Amended by | Effective date | Approved by     |                      |               |
|--------------|--|---------------|------------------------|--|------------|----------------|-----------------|----------------------|---------------|
|              |  |               |                        |  |            |                | Artwork manager | Marketing department | RA Department |
| 1            |  | N/A           | New Issue              | N/A                                      |            |                |                 |                      |               |
| 2            |  | 08-06-2021    | Updating Codes         | To be aligned with the new coding system |            |                |                 |                      |               |
| 3            |  | 16-06-2022    | to be aligned with Doc | To be aligned with the new coding system |            |                |                 |                      |               |



Amria 23512, Alexandria - Egypt  
 www.pharmaplast-online.com

Brochure artwork template no: I-TM /AR/001-6 (Version 3 )

Customer : Pharmaplast

Product name : Pharmagel Family

Issue date : 13-04-2020

Prepared by : Marina Medhat

Colors :

Dimension : A4