

# Cooling and Soothing Gel Pads

**Description:**

Cooling and Soothing Gel Pads are sticky gel patches that are applied to provide rapid cooling relief. Being moisture-rich, they hydrate and quickly absorb body heat leaving the user's skin easily soothed and cooled.

**Features and benefits:**

- Drug free, menthol free.
- Nontoxic.
- No strong odor nor burning sensation.
- Painless removal with no sticky residue
- Convenient and portable.
- work well both indoors and outdoors.

**Mode of action:**

The gel patches create a nice cooling sensation through evaporating water in the patch due to the heat of the affected skin region, allowing it to successfully retain a cool feeling.

**Intended purpose:**

Cooling and protection of affected skin area.

**Ingredients:**

- Carrier material: non-woven fabric
- Water, glycol, polymer

**Available in 2 variants:****Fevogel:**

Flexible gel patch that can fit any hotspot on the body to help lower body temperature through providing rapid cooling relief.

**Indications:**

- Relief Fever and heat strokes in summer.
- Migraine, Headache and toothache pain.
- Ease tense muscles.

**Instruction for Use:**

- Remove the plastic film and apply the hydrogel side of the pad to the desired area.
- If needed, you may secure patches in place with bandage, wrap or tape.
- No refrigeration is required, however for an added cool sensation, the breast gel pads can be stored in the refrigerator prior to use.

**Precautions:**

- The Patches are non-sterile.
- For external use only.
- Due to high water content, condensation in the packet is normal.
- Disposed of in household waste. Disposal should follow disposal procedures according to applicable local laws.

**Side effects:**

- Do not use in case of sensitivity to any of the patch ingredients.

- Amria - Alexandria - Egypt

Tel. : 002 03 4500264 Fax : 002 03 4500263

- Kafr-El-Zayat P.O.Box 38 Egypt

Tel. : 002 040 2545643 Fax : 002 040 2545645

E-mail :sales@pharmaplast-online.com

 **Pharmaplast**

www.pharmaplast-online.com

# Cooling and Soothing Gel Pads

## Cure-Aid Breast Pad:

Gel breast pads with a uniquely developed shape that fits nicely over nipples and protect them from friction with clothing for added comfort. The gel pads have special notches in the center, to allow the pad to perfectly conform to the nipples to the greatest extent it can.

## Indications:

*Cure-Aid Breast Pad are suitable for breastfeeding women:*

- To cover and protect sore nipples with superficial wounds.
- For cooling the nipples.

## Instructions for Use:

- Remove the plastic film and apply the hydrogel side of the pad over the skin with the nipple centered between the notches.
- No refrigeration is required, however for an added cool sensation, the breast gel pads can be stored in the refrigerator prior to use.

## Precautions:

- It is not necessary to wash the breasts before breastfeeding.
- When breastfeeding, gently remove the gel pads from the nipple and lay them, gel side up, on a clean surface, and they should be placed on the same side of the breast to avoid cross infection.
- The pads can be used up to 24 hours.
- The pads need to be thrown away right away if they get dirty.
- The pads are non-sterile.
- For external use only.
- Due to high water content, condensation in the packet is normal.
- Disposed of in household waste. Disposal should follow disposal procedures according to applicable local laws.

## Contraindications:

- Not Suitable for sore nipples where the dermis is severed.
- Should not be used on open wounds.
- If irritation, discomfort or infection occurs, consult your physician.
- Diabetics should consult their physician about blisters or wounds before use.

FEVOGEL	Size	Units/Pouch	Pouches/ box	Boxes/Case
FG40110	40x110 mm	1	4	24
FG50120	50x120 mm	1	4	24
FG50140	50x140 mm	1	4	24
FG100140	100x140 mm	1	4	24
CURE-AID BREAST PADS	Size	Units/Pouch	Pouches/ box	Boxes/Case
CUREBP7075	7cmx7.5cm	2	2	24
		2	4	24
		2	6	24

- Amria - Alexandria - Egypt  
 Tel. : 002 03 4500264 Fax : 002 03 4500263  
 - Kafr-El-Zayat P.O.Box 38 Egypt  
 Tel. : 002 040 2545643 Fax : 002 040 2545645  
 E-mail : sales@pharmaplast-online.com

 **Pharmaplast**  
 www.pharmaplast-online.com

Pharmaplast Amria 23512, Alexandria - Egypt www.pharmaplast-online.com		Brochure artwork template no: I-TM /AR/001-6 (Version 3)						
Customer : Pharmaplast			Product name : silotull & Silotull ONE					
Issue date : 24-02-2024			Prepared by : Nader Nabil Zaki					
Colors : CMYK			Dimension : A4					
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					