

# Cure-Aid<sup>®</sup> Nose Plaster



## Description:

**Cure-Aid<sup>®</sup> Nose Plaster** is antisnoring plaster that assists in keep airway open when applied over the nose.

## Features:

- Instant relief.
- Long lasting relief.
- Safe to use with any medicine.
- Comfortable.

## Intended Purpose:

- **Cure-Aid<sup>®</sup> Nose Plaster** is indicated for relieving of Nasal congestion associated with Allergies, Colds and deviated nasal septum.
- **Cure-Aid<sup>®</sup> Nose Plaster** elevates snoring.

## Contraindications:

- For external use only.
- Do not use over sores, sunburned or irritated skin.
- Do not use more than 12 hours per day.
- Not indicated for children under 5 years of age.
- Do not use in case of know allergy to the dressing or any of its components. If rash or skin irritation occurs, discontinue use.
- **Cure-Aid<sup>®</sup> Nose Plaster** is not intended to treat sleep apnea.

## Precautions:


- For single use only.
- Keep out of children's reach.
- Packaging contains natural rubber latex which may cause allergic reactions.

## Instructions for Use:

- Wipe the nose so as to get rid of any dirt or fats which will affect the adhesion.
- Peel off the wrapper.
- Remove the first silicon flap and stick to nose side.
- Remove the second silicon flaps and stick the plaster to the other side of the nose.
- By hand ensure that plaster is firmly adhered to the nose.

## Sizes

Cure-Aid Nose Plaster	Size	Pieces/Box	Boxes/Case	Case Dimensions
CUREN1967	19 x 67 mm Anti-Snoring	10	48	23.5x13.3x26 cm

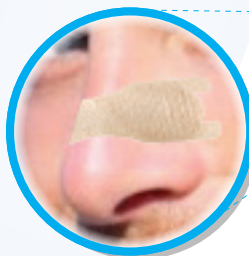

- Amria - Alexandria - Egypt  
 Tel. : 002 03 4500264 Fax : 002 03 4500263  
 - Kafr-El-Zayat P.O.Box 38 Egypt  
 Tel. : 002 040 2545643 Fax : 002 040 2545645  
 E-mail : sales@pharmaplast-online.com

 **Pharmaplast**  
[www.pharmaplast-online.com](http://www.pharmaplast-online.com)

Pharmaplast Amria 23512, Alexandria - Egypt www.pharmaplast-online.com			Brochure artwork template no: I-TM /AR/001-6 (Version 3)					
Customer : Pharmaplast			Product name : Cure-Aid Nose Plaster					
Issue date : 02-04-2016			Prepared by : Nader Nabil zaki					
Colors :			Dimension : A4					
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					
2	24-05-2022	New Text new codes	File maintenance					