

PharmaBand Skin Traction Kit



Description:

A ready to use kit provides non-invasive traction for non-operative fracture or post-operative limbs, including all items needed for the safe application of sustained traction weight support and suitable for use with a variety of traction methods.

Components of the kit:



- Fabric strapping coated with hypoallergenic hot melt adhesive.
- Foam padding attached to the middle of the strap for cushioning effect at the malleolar region.
- Plastic spacer block (suitably drilled for cord attachment).
- Elastic non adhesive bandage.

How to use:

1. Clean the limb with soap and water, prior to the application, the skin should be painted with friar's balsam (Tinct. Benz. co.)
2. The adhesive strapping is applied to the inner side of the leg, beginning from just under the knee, the unrolled further to allow placement of the foam at the malleolar region and the spacer under the heel, and finally the strapping is applied to the outer side of the leg.
3. Creases and folds should be avoided at the adhesive strapping to prevent skin blisters. In case of creases, the crease should be lifted up, slit and overlapped.
4. Apply the overlying elastic bandage spirally, overlapping by half

Features:

- High performance elastic bandage reinforces retention and aids effective traction
- High conformability for easy application
- Good skin friction characteristics
- Low allergy adhesive
- Breathable foam liner to cushion and protect bony prominences
- Available in 2 sizes, (Adult and Child).

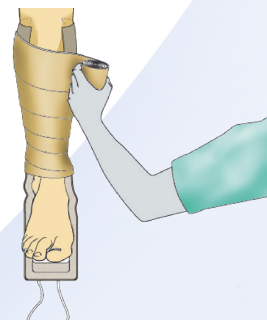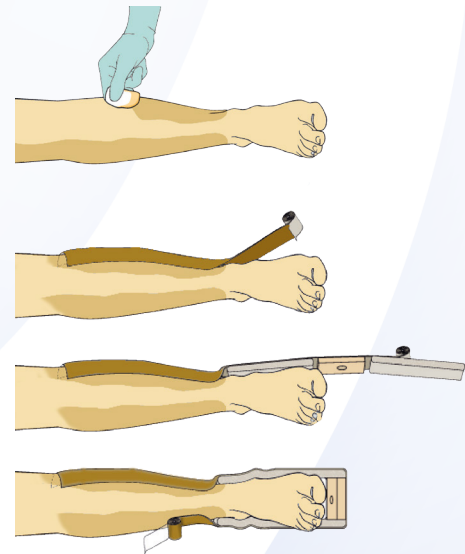

PharmaBand Skin Traction Kit

Indications:

- PharmaBand Skin traction kit used to gain and maintain bone alignment, in the case of a fracture, dislocation and to correct mild deformity.
- Low Allergy Adhesive PharmaBand Skin Traction Kit for long term effective traction.

Contraindication:

Do not apply PharmaBand skin traction kit to skin with abrasions, lacerations, surgical wounds, ulcers, loss of sensation or peripheral vascular disease.

Size	Code	Pcs/Box	Box/Case
Adult	TRACTIONADULT	1	24
Child	TRACTIONCHILD	1	24

- Amria - Alexandria - Egypt
Tel. : 002 03 4500264 Fax : 002 03 4500263
- Kafr-El-Zayat P.O.Box 38 Egypt
Tel. : 002 040 2545643 Fax : 002 040 2545645
E-mail : sales@pharmaplast-online.com

 **Pharmaplast**
www.pharmaplast-online.com

Pharmaplast		Brochure artwork template no: I-TM /AR/001-6 (Version 3)						
Customer : Pharmaplast		Product name : Skin traction kitt						
Issue date : 10-06-2020		Prepared by : Marina Medhat						
Colors : <input type="text"/>		Dimension : A4						
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					
2	10-06-2021	editing codes	File maintenance					