


Pharmapad Carbon Silver

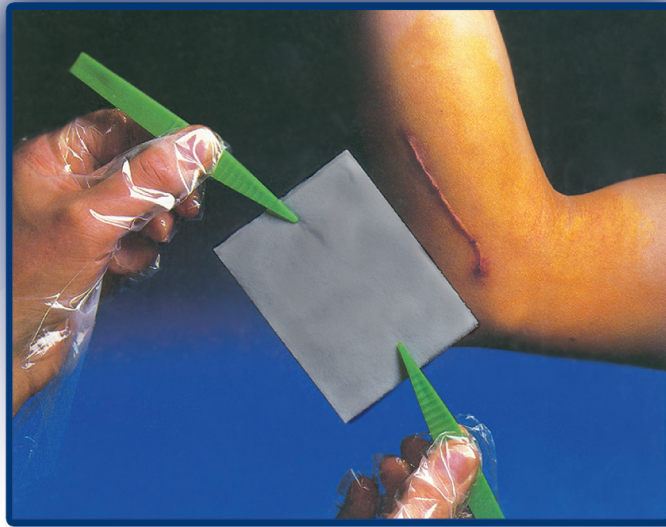

Description:

Pharmapad Carbon Silver is a thin dressing containing activated charcoal layer which is the ideal solution for malodorous wounds.

The wound contact layer is laminated with polyethylene net containing silver to provide antimicrobial effect.

Features:

- Provide an effective barrier to bacterial penetration.
- Activated Charcoal adsorbs offending odours resulting from wounds by trapping bacteria and bacterial toxins
- Antimicrobial effect of silver reduces risk of infection
- Absorbent nonwoven pad gives fast and controlled absorption of light exudates, constructed to avoid strike through.
- Individually packed in a peel pouch and EO sterilized
- Can be cut to size
- Used only as primary dressing because polyethylene silver net works only in contact with wound exudates

Pharmapad Carbon Silver

Indications:


- Pressure Ulcers
- Surgical wounds
- Diabetic Ulcers
- First and second degree burns
- Venous Ulcers

Contraindications:

Sensitivity to any of the product components

Instructions for use:

- Pharmapad Carbon silver could be used as it is or cut to size.
- Applied directly to the wound surface and covered with a secondary dressing such as Protectfilm in case of light exudates or Pharma-Foam in case of moderate or heavy exudates. It can also be held in position with a tape or fixation bandage.

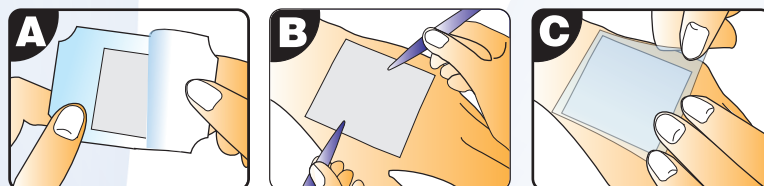

Frequency of change:

Up to 3 days according to intensity of exudates

Product code	Size	Pieces/ box	Boxes / Case
PadC Ag 5050	5 cm X 5 cm	100 Pcs	16
PadC Ag 5075	5 cm X 7.5 cm	100 Pcs	16
PadC Ag 7510	7.5 cm X 10 cm	100 Pcs	16
PadC Ag 7515	7.5 cm X 15 cm	50 Pcs	16
PadC Ag 7520	7.5 cm X 20 cm	40 Pcs	16
PadC Ag 1010	10 cm X 10 cm	50 Pcs	16
PadC Ag 1015	10 cm X 15 cm	50 Pcs	16
PadC Ag 1020	10 cm X 20 cm	40 Pcs	16
Pad C Ag 5010	5 cm X 10 cm	100Pcs	16

- Amria - Alexandria - Egypt

Tel. : 002 03 4500264 Fax : 002 03 4500263

- Kafr-El-Zayat P.O.Box 38 Egypt

Tel. : 002 040 2545643 Fax : 002 040 2545645

E-mail :sales@pharmaplast-online.com

 **Pharmaplast**

www.pharmaplast-online.com