

Kemagel[®] Amorphous

Sterile healing gel



Description:

Kemagel[®] hydro-active colloid formulation is an amorphous hydrogel which gently re-hydrates necrotic tissue, facilitating autolytic debridement, while being able to loosen and absorb slough and exudate. It can also be used to provide the optimum moist wound management environment during the later stages of wound closure.

Features:

- **Debriding action:** **Kemagel[®]** rehydrates necrotic tissue and aids debridement.
- **Desloughing action:** **Kemagel[®]** absorbs slough and exudate without damaging fragile granulation tissue.
- **Creates moist wound healing environment:** **Kemagel[®]** helps to create an optimal moist wound management environment which promotes healing, reduces the likelihood of scarring and helps soothe the pain.
- **Versatile action:** **Kemagel[®]** can be used for a dry or wet wound.
 - In case of a dry wound, **Kemagel[®]** turns the dry wound into a moist one (hydrogel effect). The dry wound absorbs the water contained in **Kemagel[®]** and promotes the dissolution of necrotic/damaged tissue.
 - In case of a wet wound, the hydrocolloid particles of **Kemagel[®]** are activated and consequently **Kemagel[®]** absorbs excessive fluid from the wound and restores the water balance in the wound (hydrocolloid effect).

Intended Purpose:

Kemagel[®] is used to create a moist wound environment for the treatment of minor conditions such as minor burns, superficial lacerations, cuts and abrasions (partial thickness wounds) and skin tears.

Kemagel[®] under the direction of a healthcare professional is used to create a moist wound environment for the management of:

- Venous ulcers (leg ulcers)
- Surgical incisions
- Diabetic foot ulcers
- Pressure ulcers (including stage IV)

Kemagel[®] creates a moist wound environment which assists in autolytic debridement of wounds covered with necrotic tissues.

Product	code	Size	Pieces /Box	Box /Case	Case Dimensions
Kemagel [®] amorphous Sachet	GELS6	6gm	100	10	29x12x10.5 cm
Kemagel [®] amorphous Sachet	GELS4	4gm	100	10	29x12x10.5 cm
Kemagel [®] amorphous Tube	GELT15	15gm	1	48	30x13x11 cm
Kemagel [®] amorphous Tube	GELT20	20gm	1	48	
Kemagel [®] amorphous Tube	GELT25	25gm	1	48	
Kemagel [®] amorphous Tube	GELT30	30gm	1	48	21x20x18 cm
Kemagel [®] amorphous Tube	GELT50	50gm	1	24	
Kemagel [®] amorphous Tube	GELT75	75gm	1	24	20x19x17 cm
Kemagel [®] amorphous Tube	GELT85	85gm	1	24	

- Amria - Alexandria - Egypt
 Tel. : 002 03 4500264 Fax : 002 03 4500263
 - Kafr-El-Zayat P.O.Box 38 Egypt
 Tel. : 002 040 2545643 Fax : 002 040 2545645
 E-mail : sales@pharmaplast-online.com

 **Pharmaplast**
www.pharmaplast-online.com

Pharmaplast Amria 23512, Alexandria - Egypt www.pharmaplast-online.com		Brochure artwork template no: I-TM /AR/001-6 (Version 3)						
Customer : Pharmaplast		Product name : Kemagel Amorphous						
Issue date : XX-XX-XXXX		Prepared by : Nader Nabil Zaki						
Colors : CMYK		Dimension : A4						
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					
2	15/10/2018	adding new variants in table	Essential requirement					
3	03/12/2018	Kemagel [®] amorphous dispenser removed from table of sizes by Dina Aziz remove the Instructions for Use and Precautions sections	Essential requirement					
4	23-05-2019	changes in Table of sizes	CRS Request, to reduce weight of shipper case					
5	30-01-2022	Updating table of sizes to be aligned with the new coding system	file maintenance					
6	03082022	adding new size	file maintenance					