


Cracked skin is a common symptom of skin dryness. Also, it occurs as a result of scratches, trauma and infection. When the skin dries, it becomes rough and flaky with small tears. These small tears may lead to deeper cracks called fissures which eventually extend to the deeper layers of the skin.

What Causes skin to crack?


- Prolonged standing.
- Obesity or being overweight.
- Open-backed shoes.
- In certain diseased conditions: For example, neuropathy in diabetic persons, which limits sweating.
- In certain skin diseases: As psoriasis and eczema.

Product description:

Crack Care® is non-woven backing material coated with sticky hydrogel matrix.

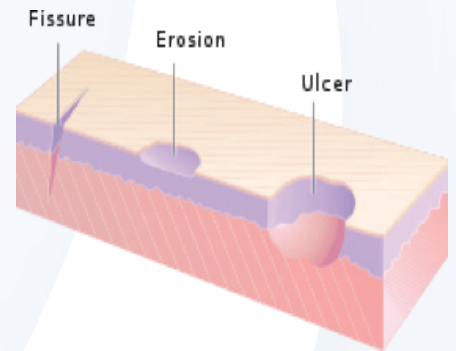

Sticky hydrogel matrix

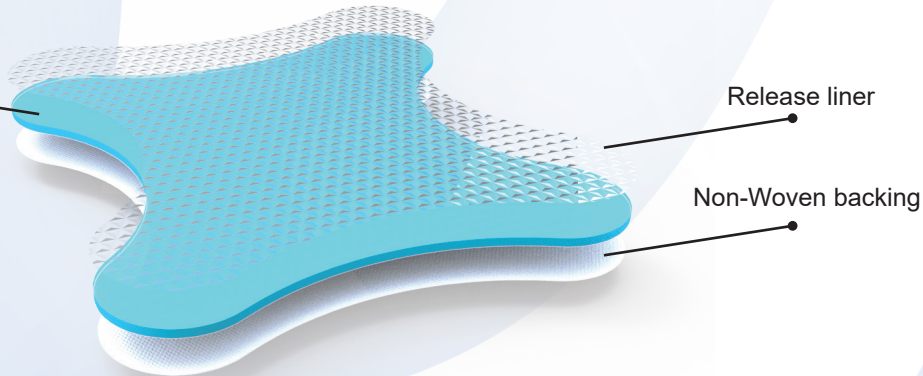

Release liner

Non-Woven backing

Precautions and Storage:

1. **Bio Secrets**® **Crack Care** is for external use only.
2. Do not apply the patch around eyes, lips, nose, mucous membrane, wounds or inflamed skin.
3. Store in a cool dry place, away from direct sunlight.
4. Keep away from the reach of children.
5. Children should use this product under adult's supervision.
6. The patch will not stick to the area if the skin is wet or oily.

Intended Purpose:

- **Bio Secrets**® **Crack Care** used to relieve discomfort conditions associated with rough, dry and cracked skin in the heel, knees and elbow.

How does Bio Secrets® Crack Care work?

- **Bio Secrets**® **Crack Care** allows the skin to be rehydrated and protected by keeping this moisturizing patch in contact with the skin overnight.
- **Bio Secrets**® **Crack Care** moisturizing ingredients provide immediate moisture, exfoliating and healing of the cracked skin.
- When **Bio Secrets**® **Crack Care** moisturizing patch is removed, it can easily be disposed which makes it a cleaner and easier method of treating dry, cracked skin than creams and lotions.



Crack Care

Instruction for Use:

1. Thoroughly wash and dry the cracked area.
2. Make sure that your skin is well dried.
3. Open the pouch and remove the release liner.
4. Apply the patch to the cracked area of skin, secure it by wearing elastic stockings or net and rub it gently.
5. keep the patch in contact with the cracked skin area overnight.

BioSecrets Crack Care	Size	Pieces/Box	Boxes/Case
BIOCC8090	80x90mm		

Other sizes are available upon request

- Amria - Alexandria - Egypt
 Tel. : 002 03 4500264 Fax : 002 03 4500263
 - Kafr-El-Zayat P.O.Box 38 Egypt
 Tel. : 002 040 2545643 Fax : 002 040 2545645
 E-mail : sales@pharmaplast-online.com



Pharmaplast Amria 23512, Alexandria - Egypt www.pharmaplast-online.com		Brochure artwork template no: I-TM /AR/001-6 (Version 3)						
Customer : Pharmaplast			Product name : BioSecrets® Crack Care					
Issue date : 01-09-2016			Prepared by : Marina Medhat					
Colors : CMYK			Dimension : A4					
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by		
						Artwork manager	Marketing department	RA Department
1	N/A	New Issue	N/A					
2	27-04-2020	amending in table of sizes	Marketing point of view					
3	29-08-2020	amending text and layer diagram, main photo	Fine maintenance					
4	23-03-2022	Updating codes	Fine maintenance					